

# Paraffin Wax Bath and Wax

## Paraffin Wax Bath

Holding up to 4 litres of Paraffin Wax, the Professional Paraffin Bath is large enough to allow the insertion of hands or feet, yet still small enough to fit on a counter top. The Temperature Display allows you to keep your Paraffin at the desired temperature so you can enjoy a relaxing and rejuvenating treatment.

- Adjustable temperature range of 30°C – 70 °C
- Includes 1 pair of Cotton Boots and Gloves
- Automatic open/close lid
- 3 pre-set modes



Code	Description	Price
PARBACAR	Caronlab Professional Paraffin Wax Bath	\$250.00

## Paraffin Wax

Specially formulated with Vitamin E, Paraffin Wax hydrates dry, rough skin, for supple, rejuvenated results. Providing both cosmetic and therapeutic benefits, the warming properties of Paraffin Wax relax muscles, ease joint pain and improve circulation.

- 800ml
- Vegan Friendly



Code	Description	Price
10PARL	Caronlab Paraffin Wax Lavender	\$22.40
10PARN	Caronlab Paraffin Wax Natural	\$22.40
10PARP	Caronlab Paraffin Wax Peach	\$22.40
10PARR	Caronlab Paraffin Wax Wild Rose	\$22.40
10PARW	Caronlab Paraffin Wax Wintergreen	\$22.40



[View Product Online](#)

Distributed by:

**Australian  
Medical Supplies**

# How To Use:

## IMPORTANT SAFETY INSTRUCTIONS

When using the Paraffin Bath, basic safety precautions should be followed, including the following:

**WARNING:** To reduce the risk of burns, electric shock, fire or injury to persons:

Read all instructions before using the appliance and keep for future reference.

1. Use this appliance only for its intended use as described in this manual.
2. Do NOT use corrosive chemicals or vapours in this appliance.
3. Do NOT operate the Paraffin Bath when empty.
4. Do NOT operate this appliance if it has a damaged cord or plug, if it is not working properly or if it has been damaged or dropped. If the supply cord is damaged, it must be replaced by the manufacturer or its service agent or a similarly qualified person in order to avoid a hazard.
5. Keep out of reach of children.
6. Close supervision is required when this appliance is in use.
7. Always use on a flat surface.
8. Do NOT place or store near water.
9. Do NOT submerge in water.
10. If the appliance falls into water, do NOT reach into the water and unplug the unit immediately.
11. Do NOT wrap the cord around the appliance when in use.
12. Always test the temperature of the Wax to avoid burns.
13. Always unplug the machine after use, allowing it to cool before handling, cleaning and storing.
14. Turn off the power source after the last use of the day.
15. **Do not heat for more than 12 hours.**

## OPERATING INSTRUCTIONS

1. Remove the Paraffin Bath from the packaging and place it on a flat surface.
2. Plug the Paraffin Bath into a power socket. The unit will make a beeping sound, and the display screen will show.
3. Press the open/close button, and the lid will automatically open. Fill the aluminium bath with the required amount of paraffin wax, then press the open/close button again to close the lid.
4. Press the power button to turn the machine on and select one of the three (3) pre-set modes.
5. Turn off the power source after the last use of the day.
6. **Do not heat for more than 12 hours.**

## MANUAL TEMPERATURE SETTING

1. Press the power button.
2. Set the temperature by pressing the or
3. The display screen will show the current temperature.

## AUTOMATIC HEATING

1. Select your heating mode by pressing the time button and using the and button to set the desired heating time based on the guidelines below.

Quantity	Heating Time	Cooling Time
450g	20 minutes	30 minutes
900g	30 minutes	45 minutes
1350g	45 minutes	1 hour

The machine will start heating after three (3) seconds. The display screen will show the countdown of heating time.

1. Once the Wax has been heated, the lid will automatically open to cool down the Wax. The display screen will show the current Wax temperature.
2. When the Wax has cooled to 52°C, the machine will continuously beep three (3) times indicating it has reached optimal temperature for use.

## HEAT DELAY MODE

1. Press the time button for three (3) seconds.
2. Press the and buttons to set the duration of time.

NOTE: Time adjustments can only be made in hourly increments.

Do not heat for more than 12 hours.

1. Display screen will show the countdown to heating time, at which point the machine will heat up to 70°C.
2. Once the Wax has been heated, the lid will automatically open to cool down the Wax. The display screen will show the current Wax temperature.

## CLEANING AND CARE

Please dispose of unused Wax carefully and wipe it clean.

Distributed by:

**Australian  
Medical Supplies**