
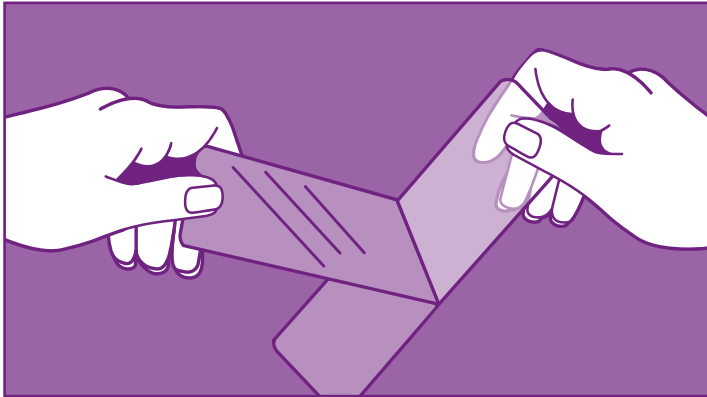


## INSTRUCTIONS FOR USE

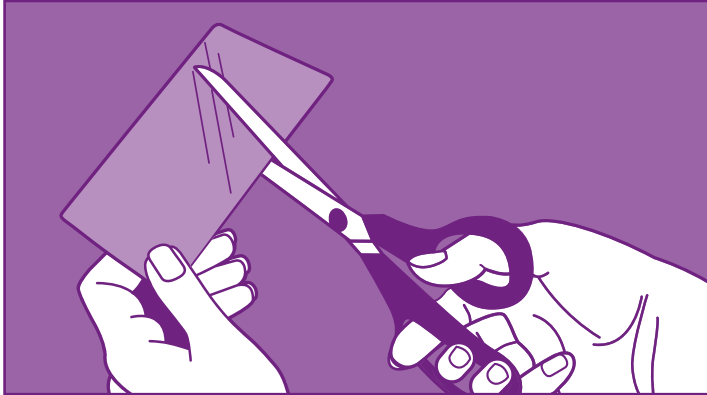

## BAPSCARCARE T SILICONE SHEETS

### Name

BAPSCARCARE T silicone sheets.

### Product description

- BAPSCARCARE T is a proven technology for the treatment of scars (hypertrophic and keloid) and for the prevention of abnormal scar development after surgery or injury.
- BAPSCARCARE T is a self-adhesive, thin, transparent silicone sheet with high adhesive strength, which ensures that it remains securely in place on any area of the body requiring treatment. Suitable for use under clothing.

### Composition

- Polyurethane film
- Medical grade silicones

### Characteristics

- BAPSCARCARE T silicone sheets ensure that scars become flatter, more flexible and thinner and that complaints (persistent redness, 'pulling' sensations, itching, restriction of joint movement, discomfort and cosmetic concerns) are reduced.
- The silicone sheet creates a semi-occlusive (covering) layer on top of the skin, which results in increased hydration of the skin (epidermis). The increased hydration ensures that damaged, scarred skin recovers more quickly and functional problems (complaints) are reduced.

### Indications/applications

BAPSCARCARE T silicone sheets are suitable for:

- both old and new scars
- all skin types
- all ages
- the treatment and prevention of abnormal scars (hypertrophic and keloid)
- the treatment of burn scars
- the treatment of surgical scars
- use under clothing and in compression therapy
- the treatment of persistent functional complaints caused by the scar (persistent redness, 'pulling' sensations, itching, restriction of joint movement, discomfort and cosmetic concerns)

### Instructions for use

- For an optimal result it is best to start using BAPSCARCARE T silicone therapy immediately after wound closure (after sutures have been removed and any scabs have disappeared).
- If necessary, the BAPSCARCARE T silicone sheet can be cut to size.
- Make sure that the silicone sheet covers the scar and a 2cm-wide area around it.
- To ensure good adhesion and durability of the silicone sheet, it is advisable to remove any hair around the scar location.
- Thoroughly clean and dry the skin before applying the silicone sheet on and around the scar.
- Remove the first part of the protective film of the silicone sheet and carefully apply the self-adhesive silicone sheet, starting at the edge of the scar. Remove the second part of the protective film and

apply the next section of the silicone sheet to the scar. Depending on the size of the silicone sheet, the above must be repeated until the entire silicone sheet is applied.

- If necessary, the silicone sheet can be fixed in place with an adhesive or elastic bandage.
- The skin may be treated with a hydrating cream after removal of the silicone sheet.

### Care instructions

- The BAPSCARCARE T silicone sheets are for single use and can be left on the scar for several days.
- Replace the silicone sheet with a new one as soon as the adhesive strength decreases significantly.

### How long should I use BAPSCARCARE T silicone sheets?

- It takes at least 3-6 months for a scar to normalise.
- You should use BAPSCARCARE T as long as the scar is raised, red or giving rise to complaints.
- Consult your therapist/doctor about the duration of therapy.

### Duration of use

- Each pack of BAPSCARCARE T contains 1 or more silicone sheets, each of which can be used for 5 to a maximum of 7 days.
- For long-term therapy, several packs may therefore be necessary.

### Special attention and precautions

- Keep out of reach of children.
- BAPSCARCARE T silicone sheets are only suitable for external use.
- Only use BAPSCARCARE T silicone sheets on healed, closed skin (after scabs have disappeared) and never on open wounds.
- Do not use if you are hypersensitive to any of the ingredients.
- Do not use if the packaging is damaged when you receive the product.

### Side effects

- In rare cases, the skin may react hypersensitively (skin irritation) to silicone treatment. In such cases, stop the treatment until the skin has calmed down.
- Start the treatment again and let the skin get used to the BAPSCARCARE T silicone sheet more gradually (one hour longer every day).
- If the skin problems persist, consult your therapist/doctor.
- If there are any serious adverse events related to the product, these should be reported to the manufacturer and the competent authorities (of the Member State) where the user is located.

### Storage

- The expiry date is stated on the packaging.
- BAPSCARCARE T silicone sheets can be stored for 5 years in its unopened packaging.
- Store at room temperature and not above 25°C.

**Date of last revision of the instructions for use**  
January 2021