


# SONORA PODIATRY CHAIR




# SPECIFICATIONS



Sonora-210, 230, 240

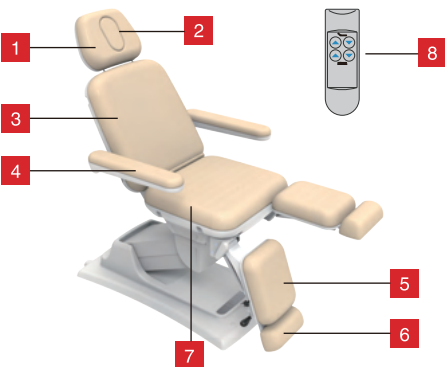

Sonora-220, 260

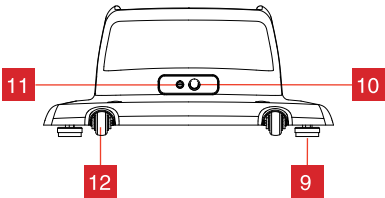

- |   |                    |    |                     |
|---|--------------------|----|---------------------|
| 1 | Headrest           | 7  | Seat                |
| 2 | Breath-hole Filler | 8  | Hand Control        |
| 3 | Backrest           | 9  | Adjustable Foot Pad |
| 4 | Side Armrest       | 10 | Power Switch        |
| 5 | Leg Rest           | 11 | Fuse                |
| 6 | Legrest Extension  | 12 | Wheels              |

## TECHNOLOGY PARAMETER

Product Name:	Sonora Beauty Bed
Model:	Sonora-210, Sonora-220, Sonora-230, Sonora-240, Sonora-260
Input:	AC 110~250V, 50-60Hz
Output:	DC 24V 1.8A
Usage Enviroment:	0-40 ℃
Power:	100W

## MAINTENANCE, CLEANING & STORAGE

- Use a damp towel to clean device slightly and dry with a soft cloth.
- Please do not use alcohol or alcohol detergent to wipe this product.
- It is allowed to use a small amount of neutral detergent to wipe the product, and any liquid shall not penetrate into the equipment.

# DISCLAIMER

---



## WARNING & CAUTION

Please read this manual before operating this equipment.

Please use this equipment according to the instructions in this brochure to avoid accidents.

Only trained beautician is allowed to operate this equipment.

DO NOT remove screws or attempt to disassemble.

Unplug the device and lock the hand control when not in use or when leaving.

---



## SAFETY INSTRUCTIONS

**DANGER: TO REDUCE RISKS OF ELECTRIC SHOCK, FIRE AND PERSONAL INJURY, THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE FOLLOWING INSTRUCTIONS:**

- For trained beautician use only.
- Before using, MUST make sure this equipment is placed on a horizontal ground. DO NOT USE when it's on an inclined surface.
- Adjust the four adjusting foot pads to make the four adjusting foot pads touch the ground smoothly, and press the four corners of the beauty bed to ensure that there is no shaking.
- This is not a toy. Please keep children away from this equipment!
- For indoor use only, do not expose the equipment to sunlight.
- Please DO NOT expose this equipment in extreme temperature or humidity environment!
- Please do not allow the patient lie on the mattress directly. Please use towel or bed sheet to protect the vinyl cover.
- Please use this equipment in the open space. Do not lift the bed or adjust the tilt angle when the path is blocked.
- Please DO NOT place anything under the table top or above the base frame. When lifting or lowered the bed or adjusting the angle of the backrest or legrest, it must be ensured that there is no obstruction of any foreign objects between the table top and the table frame.
- When lifting the bed or adjusting the angle of the backrest and legrest, make sure that the patient is firmly seated on the seat or lying on the mattress.
- DO NOT seat on the headrest, backrest, leg-rest or side armrest.
- NEVER stand or jump on the bed.
- This equipment is only used for the intended purpose or the purpose described in this manual, and it is strictly prohibited to use it for any other purposes.

## DISCLAIMER

---

- Please DO NOT disassemble or repair the equipment by uncertified personnel.
- Do not tamper or alter the equipment.
- DO NOT use this equipment in case of failure!
- Please DO NOT use water or alcohol to clean the equipment!
- NEVER put fingers, arm or any other body part under the backrest and the legrest as pinching may occur.
- Please do not immerse the equipment in water.
- When replacing the fuse, you must make sure unplug the power and turn off the power switch.

## WARRANTY

---

**IF WITHIN THREE YEAR FROM THE DATE OF PURCHASE, THIS PRODUCT FAILS DUE TO A DEFECT IN ELECTRONIC COMPONENTS OR MATERIAL OF WORKMANSHIP, AMS WILL REPAIR OR REPLACE THE PRODUCT, OR SUPPLY NECESSARY COMPONENTS FREE OF CHARGE.**

THIS WARRANTY EXCLUDES:


- a) Damage caused by accident, abuse, mishandling, or transport;
- b) Units subjected to unauthorized repair;
- c) Improper use of this device;
- d) Damage exceeding the cost of the product;
- e) Deterioration of the delivered product resulting from abnormal storage and/or safeguarding conditions on the client's premises;
- f) Failure to provide the dated proof of purchase.

# USAGE INSTRUCTIONS

---


## Power on & Power off

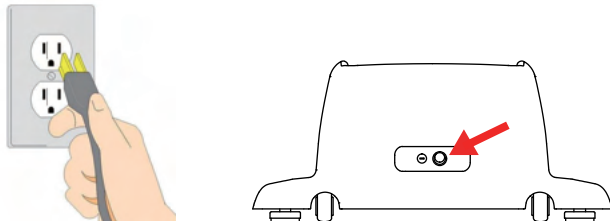

The power switch is set at the lower position on the back of the equipment. Plug the power plug into wall outlets, turn on the equipment by turn on the power switch. To turn off the equipment, turn off the power switch and pull off the power plug.



---

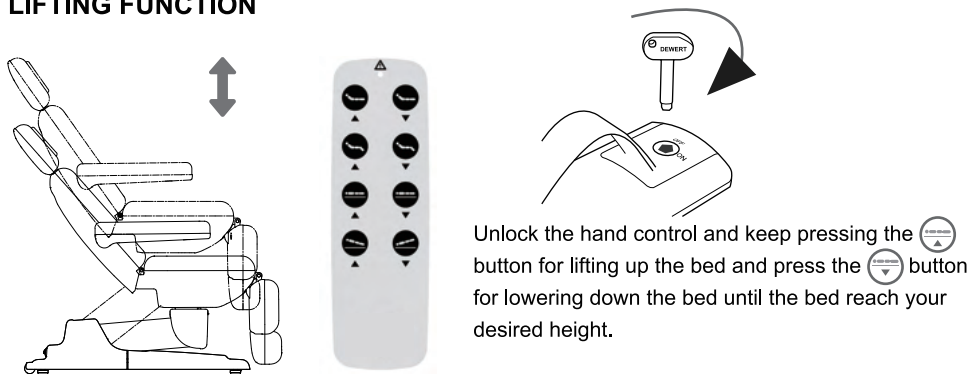
## LIFTING FUNCTION





Unlock the hand control and keep pressing the  button for lifting up the bed and press the  button for lowering down the bed until the bed reach your desired height.

---
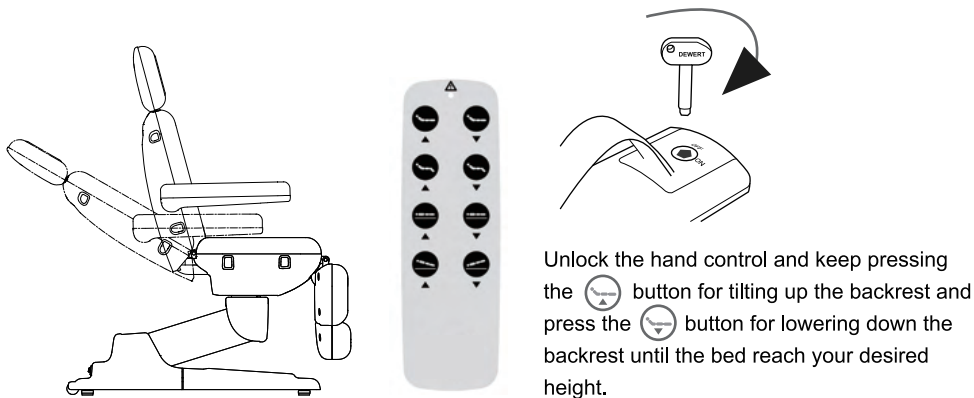
## Electric Controlled Backrest (For Sonora-220,230, 240, 260 Models)




Unlock the hand control and keep pressing the  button for tilting up the backrest and press the  button for lowering down the backrest until the bed reach your desired height.

# USAGE INSTRUCTIONS

---



## Pneumatic Controlled Backrest (For Sonora-210 Model Only)

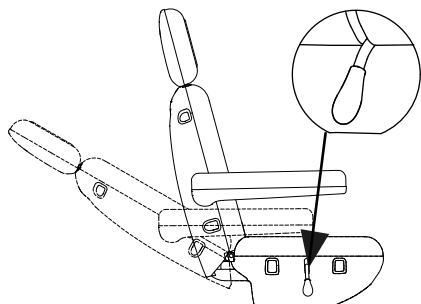



Lift up the Control Handle of the pneumatic pump, gently press down or lift up the backrest until the backrest reach your desired angle, then, release the control handle. For adjusting the backrest angle when customer is lying on the beauty bed, please make sure that the customer sits up, so as to avoid the backrest falling too fast and causing injury to the customer.

---

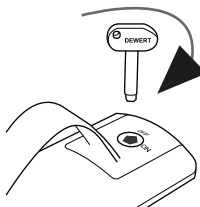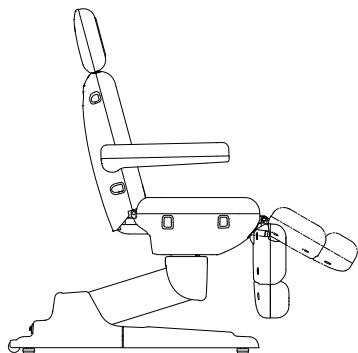
## Electric Controlled Leg-rest (For Sonora-230 & Sonora-240 Model Only)



Unlock the hand control and keep pressing the  button for tilting up the leg-rest and press the  button for lowering down the leg-rest until the leg-rest reach your desired angle.

---

## Pneumatic Controlled Leg-rest (For Sonora-220 and 260 Model Only)


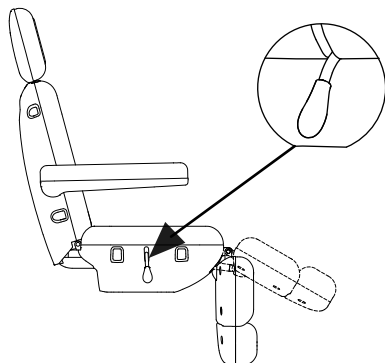


Lift up the Control Handle of the pneumatic pump, gently press down or lift the leg-rest until the leg-rest reach your desired angle, then, release the control handle. For adjusting the leg-rest angle when customer is lying on the beauty bed, please make sure that the customer lift up his/her leg, so as to avoid the leg-rest falling too fast and causing injury to the customer.

# USAGE INSTRUCTIONS

---


## Tilt Function (For Sonora-240 and Sonora 260 Model Only)

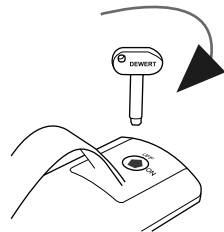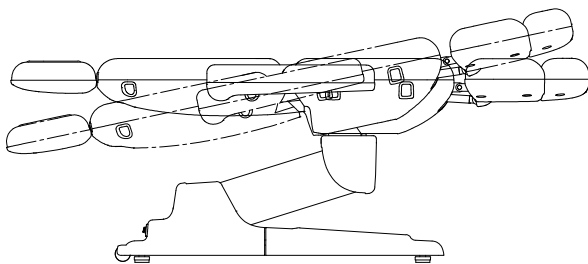

Unlock the hand control and keep pressing the  button for tilting up the table top and press the  button for tilting down the table top to your desired angle.

---


## Leg-rest Extension

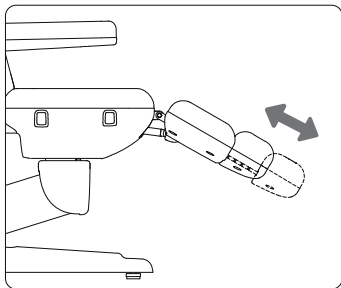

Pull the leg rest extension outward or push the leg rest extension inward to adjust the position.

---

## Adjusting the Headrest

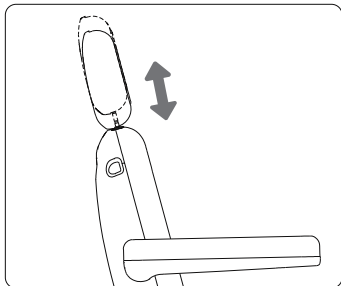


Pull the headrest outward or push the headrest inward to adjust the position.

# USAGE INSTRUCTIONS

---


## Removing & Installing the Armrest

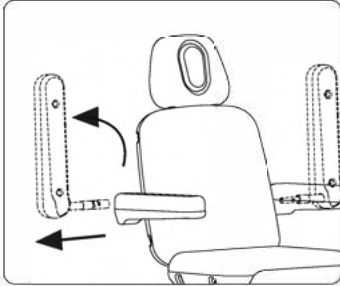

Lift the armrest up to the upright angle and pull out the armrest outwards. When installing the armrest, please keep the angle of the armrest vertical to the ground, insert the metal tube of the armrest into the round hole on the side of the backrest, and then adjust the armrest to the horizontal position.

---

## Changing the Fuse

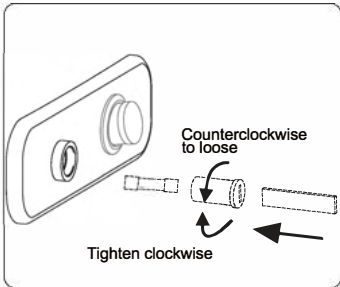

**When replacing the fuse, you must make sure unplug the power and turn off the power switch.**

In case of failure of electric lifting, please check the fuse. If the fuse is blown out, please replace it with a new one. Use a coin, metal key or thin metal piece, press the button in the middle of the fuse mounting hole and turn it counterclockwise until the fuse pops up. After installing a new 250V, 3A fuse, put it back in place, press and turn the button clockwise until it can not be turned.